



CLEANING HABA SYNTHETIC LEATHER

1) General

No solvent or products containing solvent (e.g. perchloroethylene, acetone, ...), cream cleanser products, cleaning materials with abrasive substances, cleaning materials that contain chlorine or hard brushes and sponges should be used for caring for and cleaning the imitation leather. They will damage the surface of the cushion or cover. Cleaning or care products containing oil should also be avoided, as they can make the surface brittle.

2) Daily Care/Maintenance Care

Wiping with a damp cloth is the best general care for your cover and/or cushion. For normal soiling, you can use a lukewarm neutral soap solution. Remove the soiling with a cloth and then dry the cover with a dry cotton cloth. For heavier soiling, you can also use a soft household sponge (always the soft side). Then clean with a cloth and dry off with a cotton cloth. After cleaning, the cover should be completely dry before it can be used again.

3) Use of Alcohol as a Cleaning Material

Alcohol may be used as a cleaning material up to a concentration of 70%, in particular for stubborn stains or as a disinfectant for the surface. The alcohol-containing cleaning material is dissolved in water and then used as described in 2). After cleaning, the cover should be completely dry before it can be used again.

4) Wiping Disinfectant

In cases when wiping disinfectant is required, please use the following products exclusively, as these have been tested on your cover material:

- B 30 (Oro Chemie)
- B 40 (Oro Chemie)
- B 45 (Oro Chemie)
- B 60 (Oro Chemie)

After cleaning, the cover should be completely dry before it can be used again.